



est. 1892

MarshallKloene

Designers of Custom Orthopaedic Braces,
Prosthetic Limbs and Foot Orthoses

Arrowhead Park
419 Tomahawk Dr.
Maumee, OH 43537
P: 419.893.0748
F: 419.891.9172

APPLICATION AND REMOVAL OF THE BLEDSOE KNEE BRACE

APPLICATION:

1. Lift the leg then slide the Bledsoe brace under the patient's leg.
2. Position the brace hinges with the patient's knee center.
3. Secure the four black straps.
4. While holding the medial and lateral hinges against the leg, fasten the straps directly above and below the knee. Pull the straps until they are snug. **DO NOT** over-tighten the straps. This may cause edema.
5. Secure the velcro straps at the thigh and ankle.

REMOVAL:

1. Unfasten the four black straps on the anterior portion of the brace.
2. Open the brace and slide the leg out of the brace.